



# Accel<sup>®</sup> PREvention<sup>™/MC</sup> Concentrate 48

## Concentrated One-Step Surface Cleaner & Disinfectant

Tuberculocidal, Virucidal, Bactericidal, Fungicidal, Broad-Spectrum Sanitizer

A broad spectrum, intermediate level concentrated disinfectant based on proprietary AHP<sup>®</sup> technology to deliver fast, effective cleaning performance.

### Features & Benefits

- Effective against key pathogens including MRSA, VRE, TB, Norovirus and Poliovirus, providing broad spectrum disinfection
- Virucidal, Bactericidal, Tuberculocidal and Fungicidal all in 5 minutes at 1:40. It is also a 30-second broad-spectrum sanitizer at 1:128 and a light duty cleaner at 1:256
- Compatible with most hard, non-porous surfaces while harnessing the power of AHP<sup>®</sup> technology. The AHP<sup>®</sup> active breaks down to oxygen and water after use.
- Best possible safety rating in dilution, and non-irritating to users

### Applications

- Clean and disinfect non-critical medical devices, equipment and non-porous hard surfaces





# Accel<sup>®</sup> PREvention<sup>™/MC</sup> Concentrate

Concentrated One-Step Surface Cleaner & Disinfectant

## Use instructions

### Preparation of Use-Solution

Ratio	Per Liter of Water	Per 32 oz. of Water
1:40	25 mL	0.8 oz.
1:128	8 mL	0.25 oz. (1/4 oz)
1:256	4 mL	0.125 oz. (1/8 oz)

### For use as a One-Step Surface Cleaner & Disinfectant

- Heavily soiled surfaces require cleaning prior to disinfection.
- Dilute 1:40 and apply to surface with cloth or disposable wipe.
- Let treated surface remain wet for 5 minutes.
- Air dry, wipe surfaces to dry and remove any residue, or rinse with potable water as necessary.
- Prepare a fresh solution if the solution becomes visibly dirty or diluted.

### For use as a Non-Food Contact Sanitizer

- Dilute 1:128 and apply to surface.
- Allow to remain wet for 30 seconds.
- Wipe surface and let air dry.

Prior to first use on a particular surface, it is recommended to test in a small, discreet area if there is concern of incompatibility, or contact a Diversey representative.

\* Refer to product label for complete use instructions

### 5L Closed Loop Bottle Set Up Instructions

- Remove the screw cap from Accel PREvention Concentrate 5 L bottle.
- Affix the Accel Custom Dilution System mating cap\* to the 5 L bottle (mating cap will attach to the insert inside the 5 L bottle to draw out the concentrate).
- See Accel Custom Dilution System Instructions for Use for further dilution system instructions.

\*Closed Loop Mating Cap included

Technical data	Accel <sup>®</sup> PREvention <sup>™/MC</sup> Concentrate
DIN	02448629
Certifications	EcoLogo
Color/Form	Colorless, liquid
pH	0.45 (Concentrate) 2.2 (Use Dilution 1:40)
Scent	Characteristic of ingredients
Shelf Life	2 years (Concentrate), 30 days (Use Dilution)

Product	Pack size	Dilution	Product code	
Accel <sup>®</sup> PREvention <sup>™/MC</sup> Concentrate	4 X 4 L / 1.06 U.S. Gal. Containers	1:256	100906586	🇨🇦
Accel <sup>®</sup> PREvention <sup>™/MC</sup> Concentrate	2 X 5 L / 1.32 U.S Gal. Containers	1:256	100906588	🇨🇦
Accel <sup>®</sup> PREvention <sup>™/MC</sup> Concentrate	2 x 1.5 L RTD <sup>®</sup> / 50.7 oz.	1:256	100940209	🇨🇦

## Safe handling

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use; and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDS are available online at [www.diversey.com](http://www.diversey.com) or by calling 888.352.2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate.

AHP and Design<sup>®</sup> are registered trademarks of Virox Technologies, Inc. used under license.