

Diversey

Accel®
PREvention™/MC
Wipes
One-Step Surface
Cleaner and
Disinfectant



Tuberculocidal, Virucidal, Bactericidal, Fungicidal,
Broad-Spectrum Sanitizer

A ready-to-use pre-wetted disinfectant cleaner
wipe based on proprietary AHP® technology to
deliver fast, effective cleaning performance.

Features

- One-step cleaner disinfectant
- Powered by AHP® technology
- No VOCs, no added fragrance and no NPEs
- Environmentally responsible - active ingredient breaks down to oxygen and water

Benefits

- Effective against key pathogens - including MRSA, VRE, TB, and Norovirus
- Kills Norovirus, Poliovirus, and HIV, and is Tuberculocidal, all in 3 minutes
- 30-second broad-spectrum sanitizer
- Compatible with most hard, non-porous surfaces
- Convenient pre-wetted wipes cleans, disinfects, and deodorizes in one labor-saving step

Applications

- Clean and disinfect high-touch surfaces in 3 minutes



Accel® PREvention™/MC Wipes

One-Step Surface Cleaner and Disinfectant



Use Directions: As a One-Step Cleaner Disinfectant*

- Pre-clean heavily soiled areas
- Apply disposable wipe to hard, non-porous inanimate surfaces
- All surfaces must remain visibly wet for 3 minutes
- Wipe surfaces dry, rinse, or allow to air dry

All food contact surfaces must be rinsed with potable water.

Prior to first use on a particular surface, it is recommended to test in a small, discreet area if there is concern of incompatibility, or contact a Diversey Care representative.

*Refer to product label for complete use instructions

Product Specifications

Description	
DIN	02448459
Color/Form	Colorless/Wipe
pH	2.1
Scent	Characteristic
Shelf Life	2 years

Available Items

Product Code	Description/Package Size	Dilution
100906721	12 x 160 count Wipes 15.24 cm x 17.78 cm / 6 in. x 7 in. Canisters	Ready-to-Use
100906722	12 x 60 count Wipes 25.4 cm x 25.4 cm / 10 in. x 10 in. Canisters	Ready-to-Use
100906590	120 x 10 count Wipes 19 cm x 25.4 cm / 7.5 in. x 10 in. Canisters	Ready-to-Use

Safety Reminder

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use, and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDSs are available on-line at www.sealedair.com or by calling 888 352 2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate. Please refer to the Diversey HazMat Library, only available through Internet Explorer, <http://naextranet.diversey.com/dot/>, for up to date shipping information.

AHP and Design® are registered trademarks of Virox Technologies, Inc. used under license.