

Diversey

Oxivir® Plus  
Disinfectant  
Cleaner  
Concentrate



Dilutable one-step disinfectant cleaner, hydrotherapy tub cleaner and sanitizer, formulated with proprietary hydrogen peroxide technology. Now Ecologo certified.

**Virucidal and Bactericidal**

- Disinfects in just five minutes
- Effective against Human Immunodeficiency Virus (HIV-1) (AIDS Virus), Polio virus, Pseudomonas aeruginosa, Staphylococcus aureus, Methicillin resistant Staphylococcus aureus (MRSA), Salmonella choleraesuis, Influenza A Virus and VRE
- Kills Norovirus and Canine Parvovirus
- Bactericidal - 5 minutes
- Tuberculocidal - 5 minutes
- Virucidal - 5 minutes
- Fungicidal - 5 minutes
- Non-Food Contact Sanitizer - 30 seconds at 1:128

**Compatible With Hard Surfaces**

- Active breaks down into water and oxygen by-products
- No rinsing required
- Delivers fast and effective cleaning
- Not irritating to eyes and skin in use dilution



# Oxivir® Plus

## Disinfectant Cleaner Concentrate



### Use Directions

Oxivir® Plus Disinfectant Cleaner Concentrate is a one-step disinfectant cleaner using patented AHP® technology for use on walls, stainless steel, glazed porcelain, plastic surfaces, hydrotherapy tubs and other hard, nonporous surfaces.

- To disinfect, apply a 1:40 dilution to thoroughly wet surface.
- Let solution remain on surface for a minimum of 5 minutes.
- Wipe or allow to air dry.
- Prepare a fresh solution if the solution becomes visibly dirty or diluted.

### Dilutable Disinfectant Cleaner

- 1:40 dilution – Bactericidal and virucidal, including antibiotic-resisting bacteria, in 5 minutes.
- 1:128 dilution – Non-food contact sanitizer in 30 seconds or general purpose cleaning.

### Product Specifications

Description	Concentrate
Color/Form	Colorless liquid
DIN Reg. No.	02403684
Flash Point	> 200° F (> 93.4° C)
pH	0.8 (Concentrate) 1.9 (Use Dilution 1:40)
Scent	Characteristic
Shelf Life	2 years 30 days (Diluted)
Solubility	Complete
Specific Gravity	1.037

### Available Items

Product Code	Description/Package Size	Dilution
5919024	4 x 1 gallon / 3.78 L Containers	1:40, 1:128
5919032	2 x 1.5 L RTD® Units	1:40, 1:128
5919059	2 x 84.5 oz. / 2.5 L J-Fills®	1:40

### Safety Reminder

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use, and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDSs are available on-line at [www.sealedair.com](http://www.sealedair.com) or by calling 888 352 2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate. Please refer to the Diversey HazMat Library, only available through Internet Explorer, <http://naextranet.diversey.com/dot/>, for up to date shipping information.